
EZ-Block[®]

I A N B l o c k M a d e E a s y

Quick Start Guide

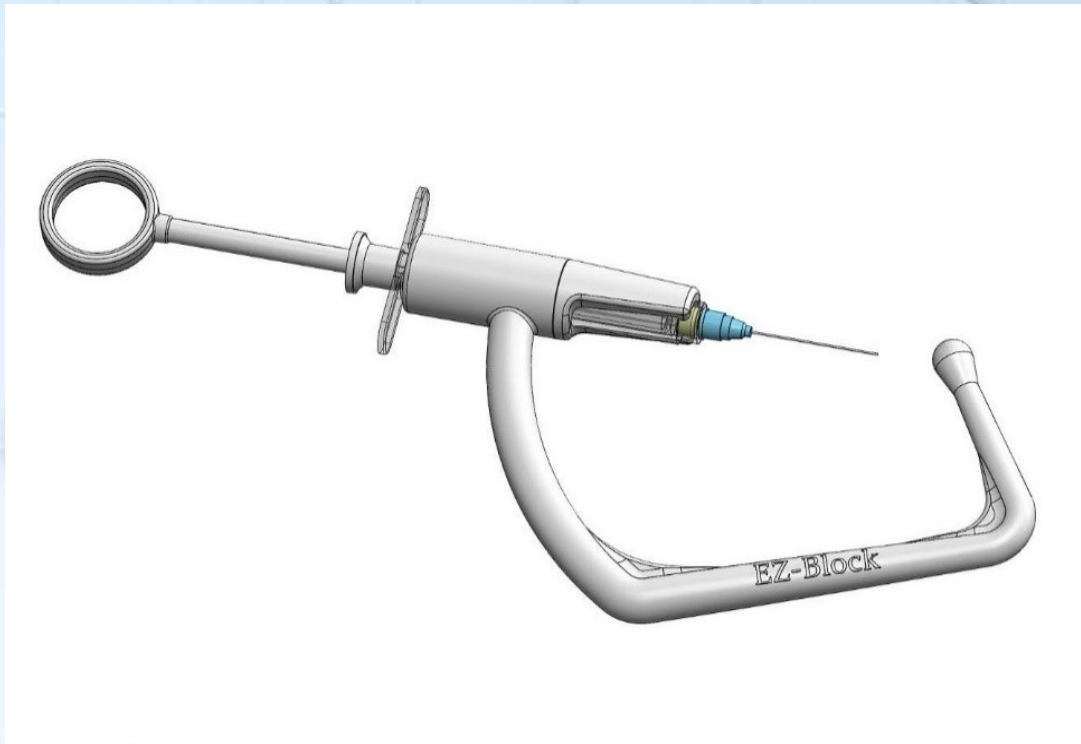Rev. 2.4 - Oct. 2022

 **Syringe positioner
for mandibular anesthesia**

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Quick Start Guide

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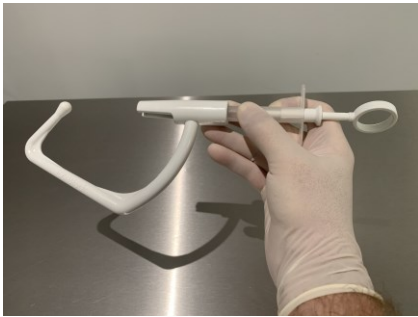
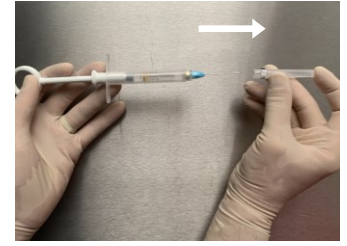
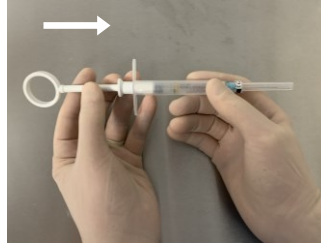
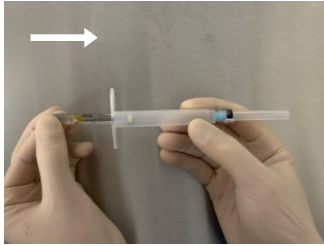
①

Insert the anesthesia cartridge into the syringe body.
 Insert the plunger as far as it will go.
 Uncap the needle.
 Insert the syringe into the tube until the tip of the needle reaches the tube end.
 Handle the EZ-Block by holding the syringe and tube like a pen.

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②

The patient in a semi-sitting position.
 For an injection on the right side, the practitioner (right-handed) stands in front of the patient.
 For an injection on the left side, the practitioner (right-handed) stands behind the patient.

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③

Ask the patient :

- to open his/her mouth wide open
- to keep the head extended and slightly turned towards the anesthesia side

It is important that the patient maintains his head in this position, **VERY STATIC**, throughout the procedure.

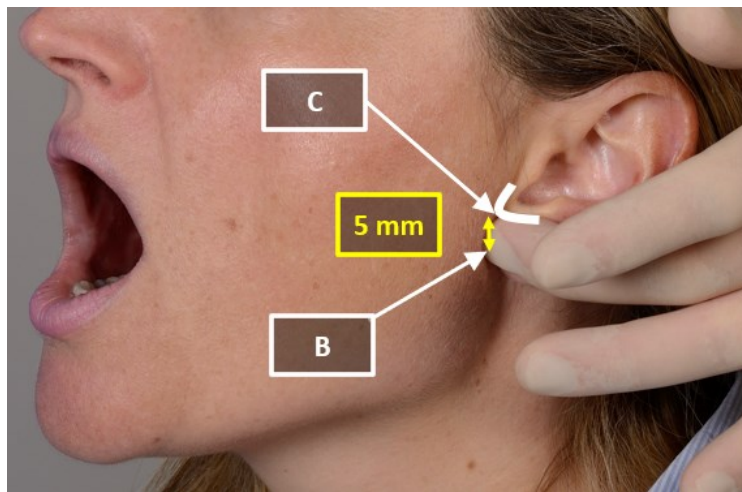
Palpate the posterior edge of the mandibular ramus (B), 5mm below the ear inter-tragic notch (C), in the **hollow** formed by the protrusion of the condyle when the mouth is opened.

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④

Insert the tube of the EZ-Block in the mouth wide open of the patient.

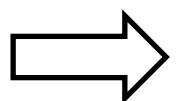
5mm below the ear inter-tragic notch, press and firmly hold the tip of the EZ-Block against the posterior edge of the mandibular ramus, in the previously palpated **hollow**.

The tip of the EZ-Block should **NOT** be positioned behind the earlobe.

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⑤ Press the tube against the occlusal surfaces of the contralateral upper premolars.

If the patient is edentulous, the guide tube is placed 1 cm below the maxillary edentulous ridge and oriented as laterally as possible.

After a visual check (for some patients, the penetration point may look higher than the point with freehand technique (A)), push the syringe in a single step until bone contact or until it stops against the tube inlet.

If bone contact is obtained prematurely, straightening the axis of the tube toward the canine allows the needle to be pushed deeper.

Inject a cartridge slowly, after performing the aspiration test. In case of pulpitis, periapical infection, or stressed patient, a second injection is sometimes necessary following the same procedure.

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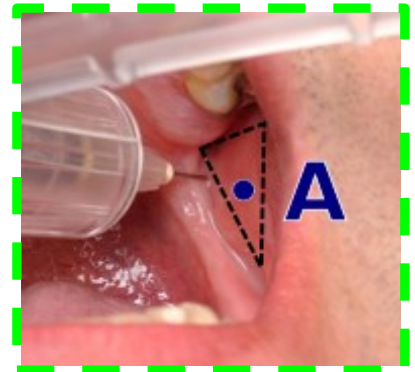
⑥ Syringe bodies are single use. They must be disposed in an adapted collector (IMW – infectious medical waste) without recapping the needle.

The EZ-Block and the plunger should be cleaned with a quaternary ammonium solution and disinfected with a glutaraldehyde solution.

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