
EZ-Block[®]

I A N B l o c k M a d e E a s y

Quick Start Guide

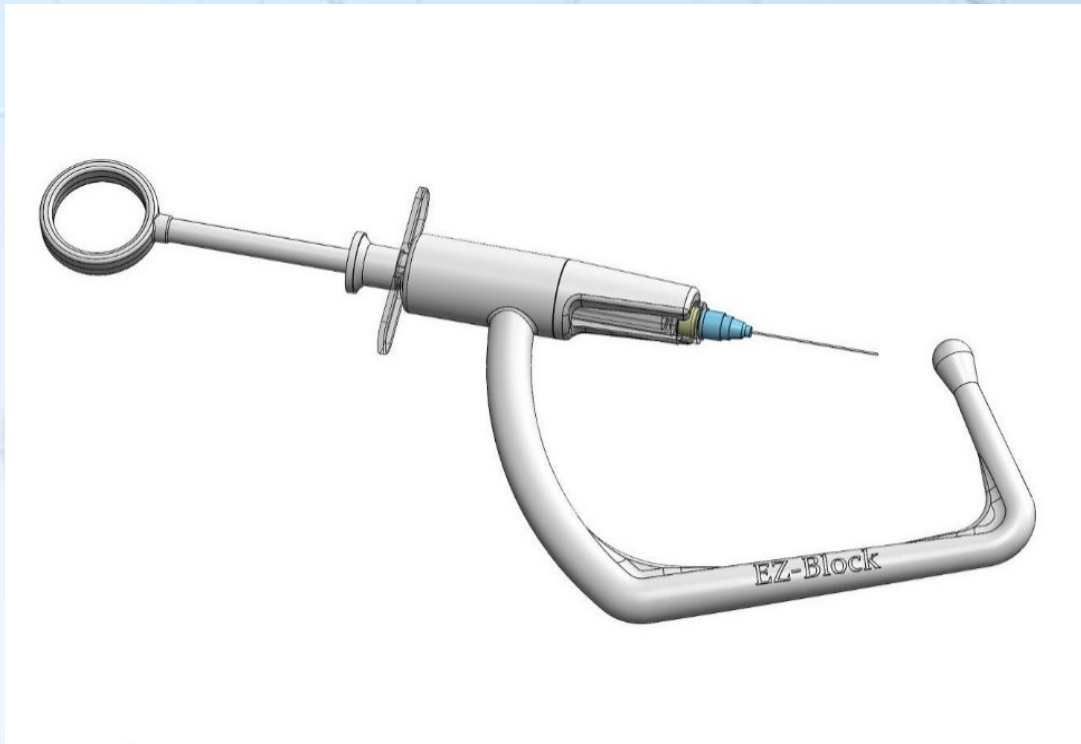Rev. 2.31 - Jan. 2021

 **Syringe positioner
for mandibular anesthesia**

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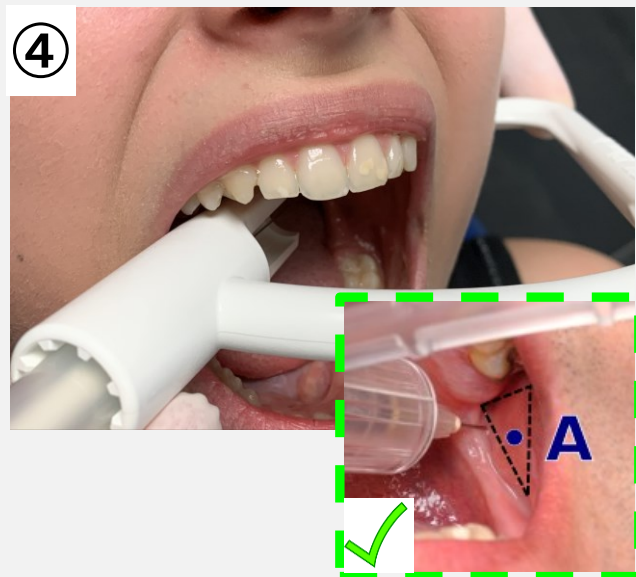
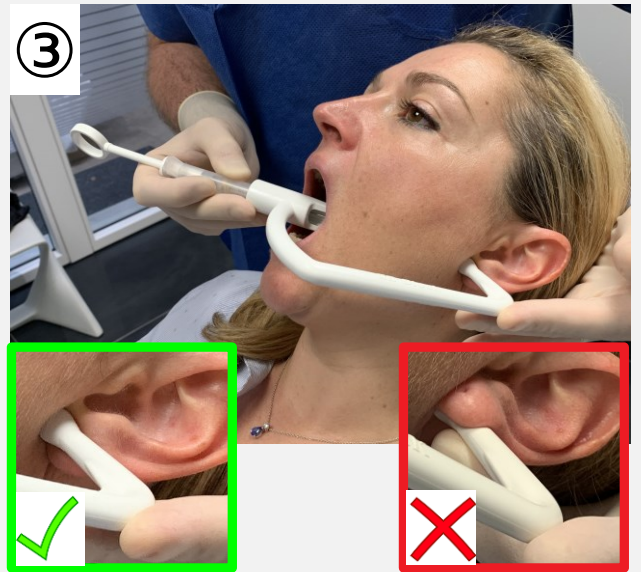
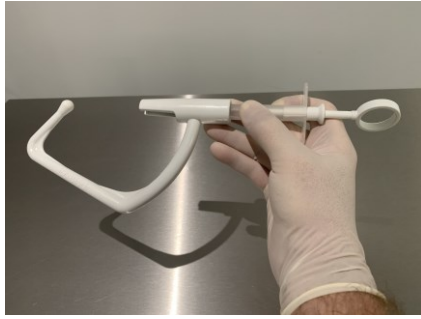
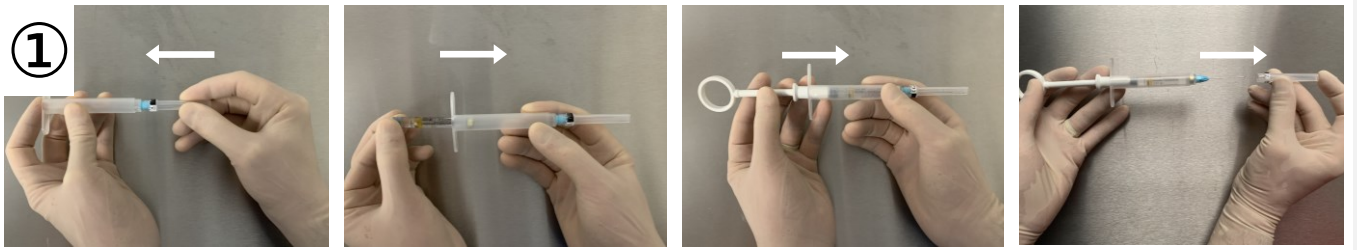
Quick Start Guide

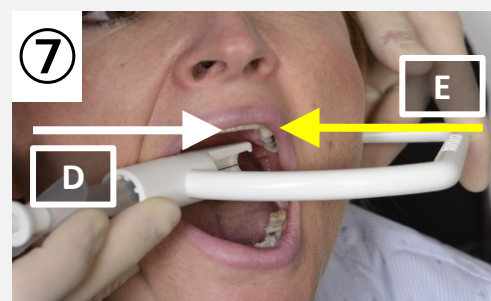
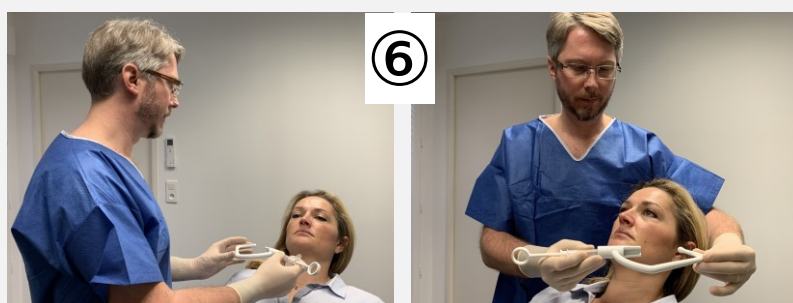
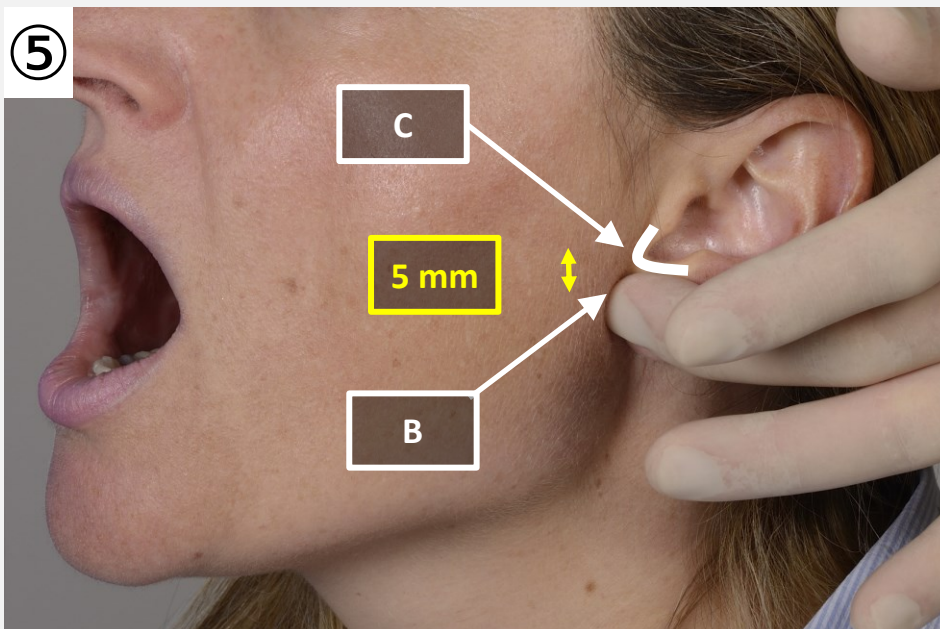
- ① Screw the needle (35 mm 25 G), then insert the anesthesia cartridge into the syringe body (Adrenaline 1/200 000).
Insert the locking sleeve as far as it will go.
Uncap the needle.
Insert the syringe into the tube **until the tip of the needle reaches the tube outlet**.
Handle the EZ-Block by holding the syringe and tube like a pen.
- ② Insert the tube of the EZ-Block in the mouth wide open of the patient.
- ③ 5mm below the ear inter-tragic notch, palpate the posterior edge of the mandibular ramus in the **hollow** formed by the protrusion of the condyle when the mouth is wide opened (see recommendation ⑤).
5mm below the ear inter-tragic notch, press and firmly hold the tip of the EZ-Block against the posterior edge of the mandibular ramus in this **hollow**.
The tip of the EZ-Block should NOT be positioned behind the earlobe.
- ④ Press the tube against the occlusal surfaces of the contralateral upper premolars.
Ensure that the patient keeps his mouth open without moving his mandible laterally.
If the patient is edentulous, the guide tube is placed 1 cm below the maxillary edentulous ridge and oriented as laterally as possible.
After a visual check (for some patients, the penetration point may look higher than the point with freehand technique (A)), push the syringe in a single step until bone contact **or** until it stops against the tube inlet.
If bone contact is obtained prematurely, straightening the axis of the tube toward the contralateral canine allows the needle to be pushed deeper.
Inject a cartridge slowly, after performing the aspiration test. In case of pulpitis, periapical infection, or stressed patient, a second injection at the same place is sometimes necessary.

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Recommendations

- ⑤ Palpate the posterior edge of the mandibular ramus (B), 5mm below the ear inter-tragic notch (C), in the hollow formed by the protrusion of the condyle when the mouth is opened.
- ⑥ The patient in a semi-sitting position, the head slightly extended.
For an injection on the right side, the practitioner (right-handed) stands in front of the patient.
For an injection on the left side, the practitioner (right-handed) stands behind the patient.
- ⑦ To stabilize the EZ-Block, ask the patient to turn his/her head towards the anesthesia side (D).
The patient should keep his/her head turned during the injection.
Hold the tip of the EZ-Block in place by applying a transverse force (E).

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